

Neater Steady-Straw (ND-KV) - Instructions

Description

- The **Neater Steady-Straw** (ND-KV) provides a stable but adjustable holder for a flexible straw.
- The non-return valve holds the liquid in the straw making sucking easier.

Set up instructions - See picture below.

- Clamp to table.
- Clip in flexible tubing.
- Bend to a convenient position for the user. **AVOID SHARP BENDS and DO NOT pull hard on the bendy arm** as this can damage it.
- Use Velcro strap to tidy set up.
- Take care that the cup used is stable enough and will not be pulled over by the straw.
- Some people may benefit from a short length of the rigid tubing being pushed all the way inside the mouthpiece end of the flexible tube.


Cleaning

- We recommend hand washing and use of sterilising solution.
- A cleaning Brush is available (ND-SC)

Safety



- Drinkers should only be used for drinking consumable liquids.
- They should not be used by people with swallowing difficulties or who are liable to choke, except under the supervision of a trained Speech and Language Therapist.
- Drinks should not be scalding hot.
- Your mouth should always be higher than the mug or siphoning can occur and cause choking.
- Clean all parts before use and check that no cracks have appeared in any components or that any parts can come loose and cause choking.
- Take care not to stick the straw in your eye or into the back of your throat.

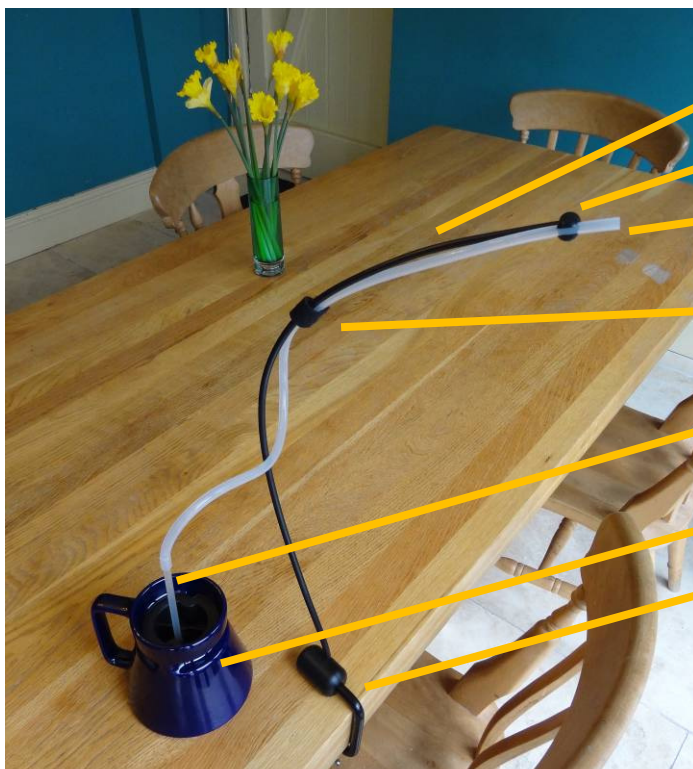

Bendy Arm

Clip

Flexible Tubing

Velcro Strap

Short rigid straw and non-return valve

Use a sturdy cup (not included)

Clamp

