

# Neater Eater Robotic (NER)

## Set-Up Manual

User's Version

CE



*Neater Solutions Ltd.*

[www.neater.co.uk](http://www.neater.co.uk)


*March 2018*

# PLEASE NOTE

Your Neater Eater Robot may have been in storage for some time and may require charging before use.

To charge:

- Connect the plug lead and the power supply that you received in the box.
- Plug the power supply into the rear of the robot.
- Place the tablet onto the front panel of the robot to charge it. Due to magnets on the tablet and within the robot, it will stick.
- A full charge will take approximately 8 hours; the robot can be used while plugged in once there is a small amount of charge in the tablet.

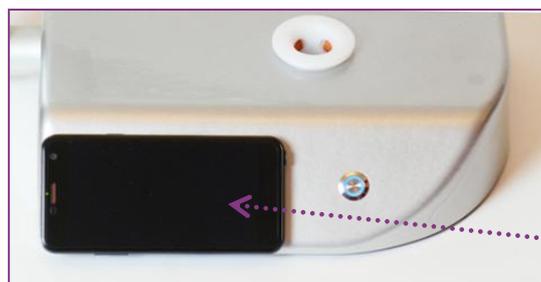


- These are the items you will find in the box of a standard Neater Eater Robotic.

**Note:** Your robot may have some different accessories depending on your order e.g. your crockery and cutlery may be different.




- This is where the tablet sits on the robot – place it here to begin.

- Switch on the robot by pressing this button on the front. It will light up blue as shown.

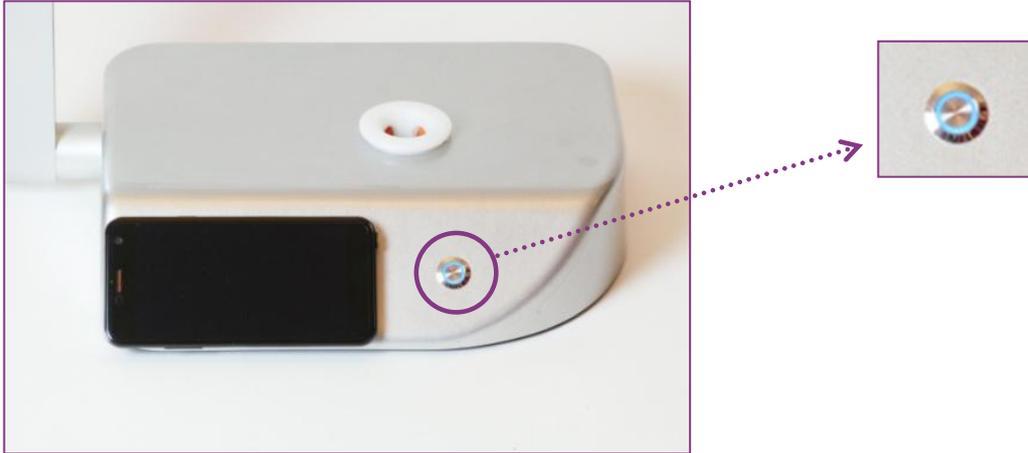

- Switch on the tablet by keeping this small button pressed for a few seconds. You will feel a small vibration when the phone is switched on; release the button.

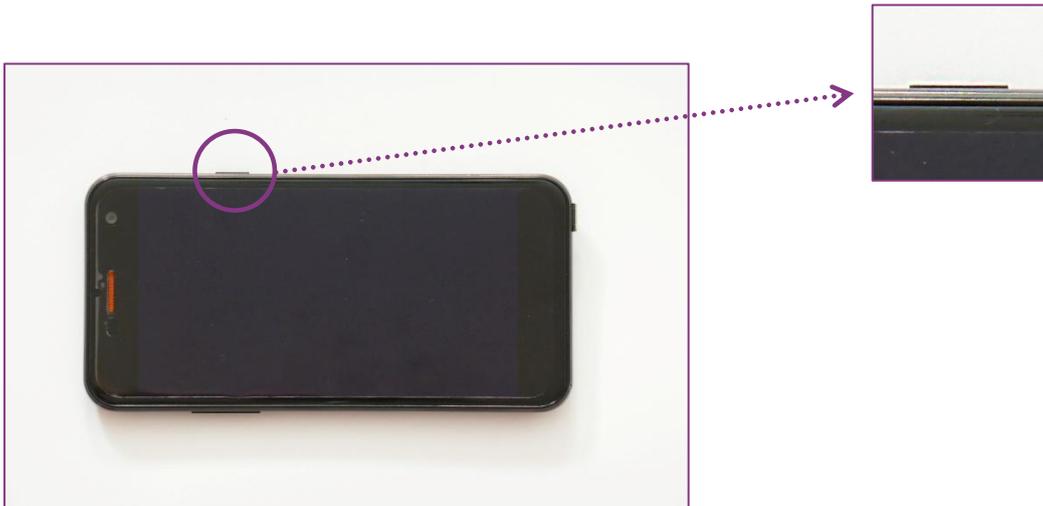


- After a sequence of displays, an introductory video will play.




- Now you will be prompted to press the question mark to read and confirm the health and safety instructions.



**Note:** You can read the instructions again at any time, by pressing the question mark icon in the top right-hand corner of the screen. It will open at the relevant section.

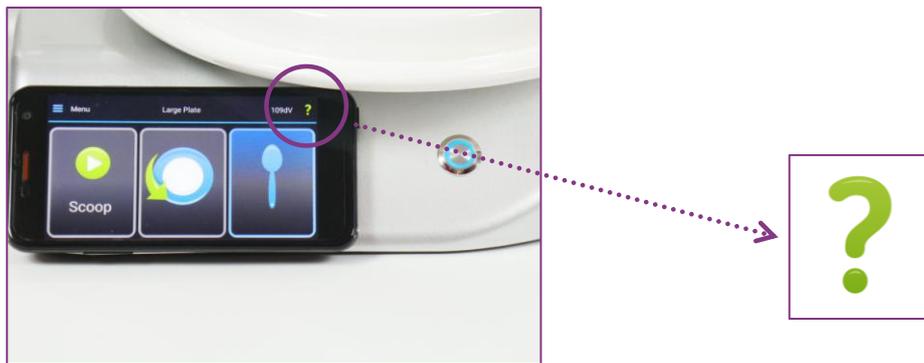

- To exit the instructions press the red cross in the corner.

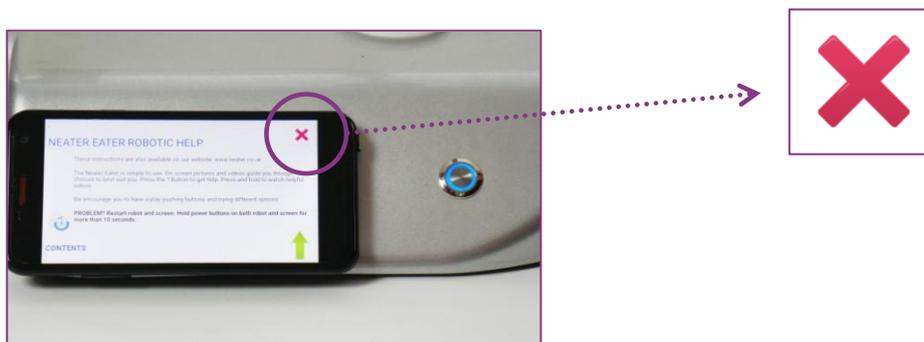

Then confirm you have read the instructions by pressing the tick icon.



- When prompted, plug the spoon onto the robot as shown here.



- The robot then moves the spoon to a pre-set position for the mouth. A screen will appear with three options, 'Continue' 'Adjust Mouth' or 'I am not user name'



- Select "Continue" if the mouth position and username are correct.
- Select "I am not user name" to select a different user.
- Select "Adjust Mouth" to teach the robot your mouth position (you will need to do this the first time you use it).
- A video animation will prompt you to move the spoon close to your mouth; you can do this by hand. Make sure you select save afterwards.

**Note:** You can change this again later by pressing the menu icon in the top left hand corner of the screen. Select the Mouth Position icon, and prompts will guide you to change the settings.



Your robot is nearly ready for you to eat with!

- When prompted, place the plate onto the robot, as shown here. Make sure it is pushed all the way down.



- You will then be presented with the option of selecting which plate you have put onto the robot. If you know which it is, you can manually select which plate you have by pressing the 'Select Plate' icon and choosing from the options as shown below.



- Alternatively, press 'Check Plate Now', and the robot will automatically do it for you.



- Ensure your food is cut into bite-sized pieces that will fit onto the cutlery you have chosen.

Press 'Scoop' to begin, and enjoy your meal!

